

### 9-3 Replacing Mist Filter

Open the door and remove the used Mist Filters. Replace with new Mist Filters as shown in Fig. 25.



#### CAUTION

- Replace immediately with new Mist Filters when signs of debris appear.
- Dispose of used Mist Filters as medical waste according to the laws and regulations in your country.

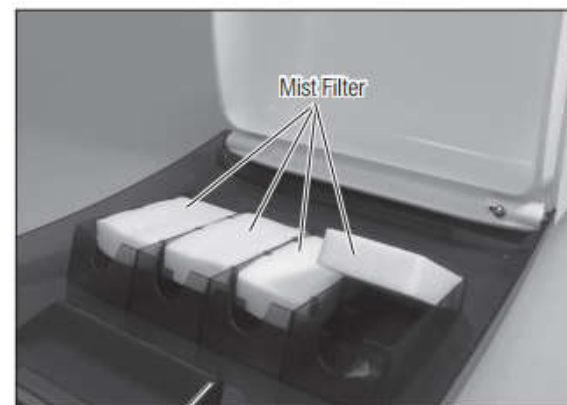


Fig. 25

### 9-4 Replacing the Oil Absorber Sheet

Open the door and remove used Oil Absorber Sheet. Place new Oil Absorber Sheet at the bottom of the lubricating chamber of the Unit as shown in Fig. 26.



#### CAUTION

- Replace immediately with a new Oil Absorber Sheet when signs of debris appear.
- Dispose of used Oil Absorber Sheets as medical waste according to the laws and regulations in your country.



Fig. 26